



SELECT COMBO II QUICKSTART GUIDE

PORTABLE MUSCLE STIMULATOR
WITH 4 WAVEFORMS

Not for use on patients that have a cardiac pacemaker, implanted defibrillator, or other implanted electronic device. Do not use on patients whose pain syndromes are undiagnosed.

STEPS FOR USE

- 1) To insert battery press down on top of the belt clip to release then remove battery cover
- 2) Press blue button to turn device on/off or to return to main screen
- 3) Press M/II button to select treatment mode: **TENS / EMS / IF / RUSS**
M/II button can also be pressed if there is an immediate need to pause during treatment
- 4) Select and press one of the therapeutic body part buttons, each one has 3 programs, P1 and P2 which are preprogramed and U1 a manual/custom mode to be set by clinician.
- 5) Press S button to enter treatment time, the minutes will flash in the screen, use the plus or minus buttons to adjust time, when done press S to confirm and return to home screen, you are now ready to begin treatment by turning up the power with the plus buttons
- 6) Safety lock – The device locks after stimulation has been on for 20 seconds and adjustments are complete, an indicator that looks like a key will be displayed on the screen, to unlock press either of the minus buttons
- 7) U1- manual / custom mode - Using the same S button that you entered treatment time in the preprogram modes you will notice after time is set and you press the S button again the next available parameter for adjustment will begin to flash, after all available parameters have been adjusted press the S button you will be back at the main screen ready to turn up power and begin treatment.

